

# MRI Scan Patient Checklist

## ICD Systems

- Ellipse™ VR ICD
- Ellipse™ DR ICD
- Fortify Assura™ VR ICD
- Fortify Assura™ DR ICD



Patient: \_\_\_\_\_

Address: \_\_\_\_\_

Device model: \_\_\_\_\_ Lead model(s): \_\_\_\_\_

## Steps for an MRI Scan<sup>1</sup> for the Cardiology Department

ICD System Identification	
Implanted ICD and leads are labeled as St. Jude Medical MRI Conditional	<input type="checkbox"/>
Confirmed that other previously implanted (active or abandoned) cardiac hardware that may include medical devices, leads, lead extenders, lead adaptor or port plugs are not present	<input type="checkbox"/>
St. Jude Medical MRI Ready system is implanted in the left or right pectoral region	<input type="checkbox"/>
St. Jude Medical MRI Ready defibrillation lead type identified as either Durata™ lead or Optisure™ lead Note: Please see the St. Jude Medical MRI Procedure Information document for appropriate lead lengths	<input type="checkbox"/>
St. Jude Medical MRI Ready defibrillation lead identifies as either 58 cm or 65 cm	<input type="checkbox"/>
Preparing Patient for MRI	
MRI settings have been selected and saved	<input type="checkbox"/>
Review the checklist on the Merlin™ programmer and perform required tests <sup>2</sup>	<input type="checkbox"/>
St. Jude Medical Merlin programmer is used to enable MRI settings	<input type="checkbox"/>
After patient receives MRI scan, disable MRI settings	<input type="checkbox"/>

Name: \_\_\_\_\_

Signature: \_\_\_\_\_ Date: \_\_\_\_\_

# Steps for an MRI Scan<sup>1</sup> for the Radiology Department

MRI Scan Conditions	
Static magnetic field strength of 1.5 Tesla (T) only	<input type="checkbox"/>
Maximum gradient slew rate of 200 T/m/s per axis	<input type="checkbox"/>
Body coil built into MRI machine used as RF transmitter (Use of local transmit-only or local transmit/receive coils has not been studied. Receive-only coils can be safely used)	<input type="checkbox"/>
Patient properly positioned (not on his or her side within the MRI bore)	<input type="checkbox"/>
Full St. Jude Medical MRI Ready system pre-approved by Cardiology Department	<input type="checkbox"/>
Continuous monitoring of the patient's hemodynamic function using at least one of the following systems: electrocardiography, pulse oximetry, noninvasive blood pressure measurements	<input type="checkbox"/>
External defibrillator available during the MRI scan	<input type="checkbox"/>
Different scan zone, whole body SAR levels, and scan time restrictions may apply – St. Jude Medical MRI Procedure Information document consulted to identify the appropriate scan exclusion zone and SAR limit depending on the lead that is used	<input type="checkbox"/>

Name: \_\_\_\_\_

Signature: \_\_\_\_\_ Date: \_\_\_\_\_

- For additional instructions, refer to the St. Jude Medical MRI Procedure information document, the Ellipse ICD, Fortify Assura ICD, Durata lead and Optisure lead manuals.
- Do not take the Merlin Programming system into the MRI room (Zone IV) or past the 5 Gauss line.

**Rx Only**  
**Brief Summary:** Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use.

Unless otherwise noted, <sup>TM</sup> indicates that the name is a trademark of, or licensed to, St. Jude Medical or one of its subsidiaries. ST. JUDE MEDICAL and the nine-squares symbol are trademarks and service marks of St. Jude Medical, Inc. and its related companies. © 2015 St. Jude Medical, Inc. All Rights Reserved.

**St. Jude Medical Inc.**  
 Global Headquarters  
 One St. Jude Medical Drive  
 St. Paul, Minnesota 55117  
 USA  
**T +1 651 756 2000 | F +1 651 756 3301**

**St. Jude Medical S.C., Inc.**  
 Americas Division  
 6300 Bee Cave Road  
 Bldg. Two, Suite 100  
 Austin, TX 78746  
 USA  
**T +1 512 286 4000 | F +1 512 732 2418**

**SJM Coordination Center BVBA**  
 The Corporate Village  
 Da Vincilaan 11-Box F1  
 B-1935 Zaventem, Belgium  
**T +32 2 774 68 11 | F +32 2 772 83 84**

**St. Jude Medical Brasil Ltda.**  
 Rua Itapeva, 538  
 5º ao 8º andares  
 01332-000 – São Paulo – SP  
 Brazil  
**T +55 11 5080 5400 | F +55 11 5080 5423**

**St. Jude Medical (Hong Kong) Ltd.**  
 Suite 1608, 16/F Exchange Tower  
 33 Wang Chiu Road  
 Kowloon Bay, Kowloon  
 Hong Kong SAR  
**T +852 2996 7688 | F +852 2956 0622**

**St. Jude Medical Japan Co., Ltd.**  
 Shiodome City Center 15F  
 1-5-2 Higashi Shinbashi, Minato-ku  
 Tokyo 105-7115  
 Japan  
**T +81 3 6255 6370 | F +81 3 6255 6371**

**St. Jude Medical Australia Pty, Ltd.**  
 17 Orion Road  
 Lane Cove, NSW 2066  
 Australia  
**T +61 2 9936 1200 | F +61 2 9936 1222**

