



ABBOTT MRI-READY SYSTEM

MRI Scan Clinician Checklist

This document is intended to be used
as a guide for health care facilities.

Patient Name: _____ Date of Birth: _____

Device Information:

Pacemaker Manufacturer: _____ Product Model #: _____
ICD Manufacturer: _____ Product Model #: _____

Lead Information:

RA: ☐ Yes ☐ No | Manufacturer: _____ Product Model #: _____ Lead Length: _____ Implant Location: _____
RV: ☐ Yes ☐ No | Manufacturer: _____ Product Model #: _____ Lead Length: _____ Implant Location: _____
LV: ☐ Yes ☐ No | Manufacturer: _____ Product Model #: _____ Lead Length: _____ Implant Location: _____

STEPS FOR AN MRI SCAN^a CARDIOLOGY DEPARTMENT

Please refer to the Abbott MRI-Ready Systems Manual at manuals.eifu.abbott

MRI SYSTEM CONDITIONS	YES	NO
Patient has a complete MRI-Ready System with MR Conditional pulse generator and lead(s).	<input type="checkbox"/>	<input type="checkbox"/>
Confirm the absence of lead extenders, lead adapters, and broken or abandoned leads. ¹ <small>^aScanning patients who have other MR Conditional devices that are not implanted in cardiac tissue is acceptable provided all MR Conditional requirements for each implanted device are met. Scanning patients who have other MR Conditional devices implanted in cardiac tissue that are at least 2 cm from the lead electrodes is acceptable provided all MR Conditional requirements for each implanted device are met. ¹Abandoned leads include, but are not limited to, implanted leads with any connector(s) not plugged into the device.</small>	<input type="checkbox"/>	<input type="checkbox"/>
Lead impedance measurements are within the programmed lead impedance limits.	<input type="checkbox"/>	<input type="checkbox"/>
Pulse generator is implanted in the left or right pectoral region. For Quadra Allure™ CRT-Ps with Tendril™ STS or UltiPace™ Pacing Leads ONLY: Lead(s) implanted in right atrium and/or right ventricle.	<input type="checkbox"/>	<input type="checkbox"/>
Pulse generator has sufficient battery (not at end-of-life EOL).	<input type="checkbox"/>	<input type="checkbox"/>
PRE-MRI SCAN DEVICE SETTINGS	YES	NO
Confirm pacing capture thresholds are stable.	<input type="checkbox"/>	<input type="checkbox"/>
Confirm capture threshold ≤ 2.5 V at a pulse width of 0.5 ms for RA and RV lead(s).	<input type="checkbox"/>	<input type="checkbox"/>
Confirm capture threshold of ≤ 2.0V at a pulse width of 0.5 ms for the LV lead.	<input type="checkbox"/>	<input type="checkbox"/>
Confirm MRI Settings status and the programmed MRI Mode setting. Please select a pacing rate to avoid competitive pacing: <div>MRI Pacing Mode <input type="checkbox"/> DOO (Bipolar) <input type="checkbox"/> AOO (Bipolar) <input type="checkbox"/> VOO (Bipolar) <input type="checkbox"/> Pacing Off</div> <div>MRI Pacing Rate <input type="checkbox"/> _____ bpm <input type="checkbox"/> Other: _____ <input type="checkbox"/> Intrinsic rate + _____ bpm</div>		
REMINDER: It is required to program the device to MRI Settings as part of the MRI scan workflow.		
For pacemakers ONLY: if the SJM MRI Activator™ handheld device is used, ^b verify that it has been enabled using the Merlin™ Patient Care System (PCS) programmer.	<input type="checkbox"/>	<input type="checkbox"/>
Print an MRI summary report of the patient's MRI Settings and permanently programmed parameters. For Gallant™, Entrant™, and Neutrino™ NxT ICDs & CRT-Ds ONLY: If MRI Timeout is programmed, ensure that there is adequate time to complete the MRI scan before MRI Timeout expires.	<input type="checkbox"/>	<input type="checkbox"/>
POST-MRI SCAN DEVICE SETTINGS	YES	NO
Disable the pulse generator MRI Settings, restoring the permanently programmed settings.	<input type="checkbox"/>	<input type="checkbox"/>
Confirm the permanently programmed settings are appropriate.	<input type="checkbox"/>	<input type="checkbox"/>
Check the pacing capture thresholds after the scan to ensure that the pacing parameters are programmed adequately for the patient based on the threshold.	<input type="checkbox"/>	<input type="checkbox"/>

Physician Name: _____ Date: _____

Physician Signature: _____

STEPS FOR AN MRI SCAN RADIOLOGY DEPARTMENT

Please refer to the Abbott MRI-Ready Systems Manual at manuals.eifu.abbott

MRI SCAN CONDITIONS	YES	NO
Confirm patient has a complete MRI-Ready System with MR Conditional pulse generator and leads(s).	<input type="checkbox"/>	<input type="checkbox"/>
Confirm that no adverse conditions to MRI scanning are present.	<input type="checkbox"/>	<input type="checkbox"/>
Select the correct Scan Parameters for your MRI equipment.	<input type="checkbox"/>	<input type="checkbox"/>
Confirm the MRI Settings status (MRI Settings must be enabled before the scan). For Gallant™, Entrant™, and Neutrino™ NxT ICDs & CRT-Ds ONLY: If MRI Timeout is programmed, ensure that there is adequate time to complete the MRI scan before MRI Timeout expires. Patient must exit MRI scanner before (time and date) _____	<input type="checkbox"/>	<input type="checkbox"/>
Perform the scan and monitor the patient.	<input type="checkbox"/>	<input type="checkbox"/>
After patient receives MRI scan, disable the pulse generator MRI Settings. Note: If you are not using the SJM MRI Activator™ handheld device, the MRI Settings must be disabled by the patient's device management physician, clinician or Abbott representative using the Merlin™ PCS programmer.	<input type="checkbox"/>	<input type="checkbox"/>
IF SJM MRI ACTIVATOR DEVICE IS USED ^b (ONLY WITH PACEMAKERS)	YES	NO
Check MRI Settings status using the SJM MRI Activator handheld device.	<input type="checkbox"/>	<input type="checkbox"/>
If MRI Settings are not enabled, enable MRI Settings before the MRI scan using the SJM MRI Activator handheld device.	<input type="checkbox"/>	<input type="checkbox"/>
After patient receives MRI scan, disable MRI Settings using the SJM MRI Activator handheld device.	<input type="checkbox"/>	<input type="checkbox"/>

Technical Support

Abbott maintains 24-hour phone lines for technical questions and support:

1-818-362-6822

1-800-722-3774 (toll-free within North America)

+46 8 474 4147 (Sweden)

+ 61 2 9936 1200 (Australia)

manuals.eifu.abbott

For additional assistance, call your local Abbott representative.

Physician Name: _____ Date: _____

Physician Signature: _____

a. For any additional information, including warnings, precautions, and potential adverse events, please refer to the Abbott MRI-Ready Systems Manual at manuals.eifu.abbott or visit cardiovascular.abbott/mrireaddy.

b. Do not bring any external devices, such as the Merlin™ PCS Programmers, SJM MRI Activator™ device or a smart phone or tablet running a mobile application, into the scanner magnet room (Zone IV).

This document is intended to be used as a guide for health care facilities.

Rx Only

Brief Summary: Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use.

[™] Indicates a trademark of the Abbott group of companies.

[‡] Indicates a third-party trademark, which is property of its respective owner.

Abbott

15900 Valley View Court Sylmar, CA 91342

Tel: +1 818 362 6822

Cardiovascular.Abbott

© 2025 Abbott. All Rights Reserved.
MAT-2205945 v3.0 | Item approved for U.S. use.

