


To turn on press the blue power button to turn on the transmitter. Choose your preferred language. Tap **START** to begin setup. Wait for setup to complete.


Note: Set up can take up to two minutes to complete. Follow directions on the transmitter screen.




While your APT loads, dampen a cloth with water. The dampened cloth will be used to prepare your skin for data transmission.

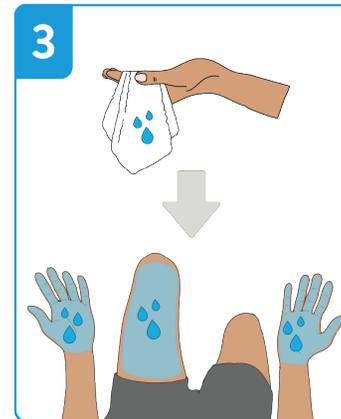


Dab the damp cloth on the palms, fingers, and left thigh.

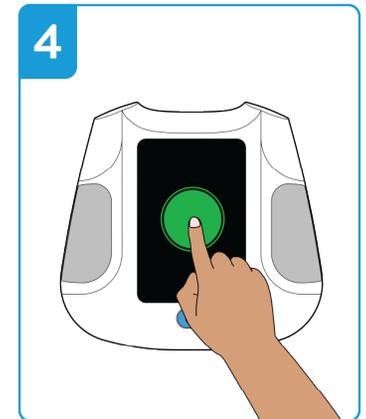

Tap the green button on the **GET READY!** screen.

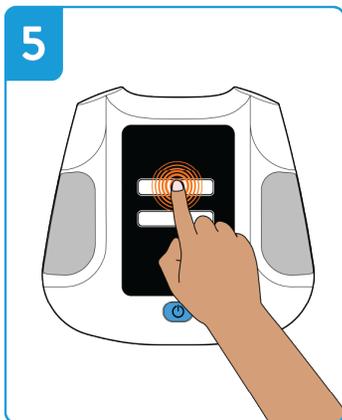


Tap **HAND-HELD** on the **SELECT USE** screen.

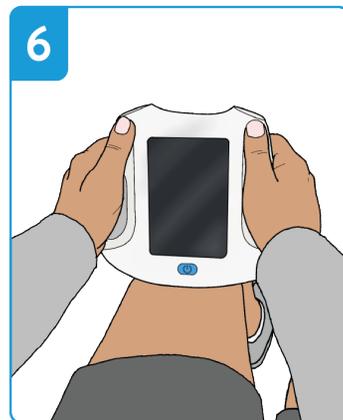

Gently grip the transmitter making sure you are in contact with the metal electrodes on the front and back of the transmitter.



While gripping the transmitter, place the bottom of the transmitter on the skin of your thigh.

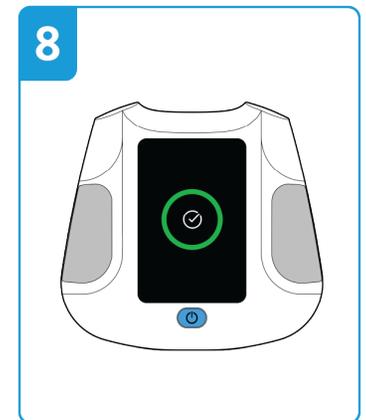

Setup is complete.

For additional assistance, please contact Technical Support:

If you have trouble using the transmitter or have any questions regarding its use, contact Technical Support Monday through Friday (8AM to 8PM Eastern Standard Time).

Call **1-877-696-3754** (toll-free within North America) or e-mail myMerlin@abbott.com



For additional information about your transmitter, please refer to cardiovascular.abbott/aveir-transmitter, scan the QR code, or contact your local Abbott Medical representative.



Rx Only

Brief Summary: Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use.

Indication for Use: The AVEIR™ Patient Transmitter is indicated for use with patients who have been implanted with compatible AVEIR leadless pacemakers (LP)(s).

Intended Use: The AVEIR™ Patient Transmitter is intended to retrieve device data from implanted AVEIR™ Leadless Pacemakers (LP)(s) and wirelessly transmit the retrieved data to Merlin.net™ Patient Care Network (PCN).

The AVEIR Patient Transmitter Wired Electrodes are intended to be used as an optional accessory component with the transmitter to facilitate retrieval of device data from implanted AVEIR LP(s).

Contraindications: There are no known contraindications for the AVEIR™ Patient Transmitter.

Adverse Events: There are no adverse events associated with the AVEIR™ Patient Transmitter.

Refer to the User's Manual for detailed indications, contraindications, warnings, precautions, and adverse events.

™ Indicates a trademark of the Abbott group of companies.

‡ Indicates a third-party trademark, which is property of its respective owner.

Abbott

15900 Valley View Court, Sylmar, CA 91342

Tel: +1 818 362 6822

Cardiovascular.Abbott

© 2026 Abbott. All Rights Reserved.

MAT-2513540 v2.0 | Item approved for U.S. use.

