


SAFETY AND COMPLICATIONS RESULTS


- The primary safety assessment met the pre-defined acceptance criterion
- Access-related major complication rates at 48 hours (1.3%) and 30 days were low (1.9%)
 - + Most complications occurred within 48 hours

Freedom From Major Femoral Vein Access-Related Complications⁴ Through 30 Days

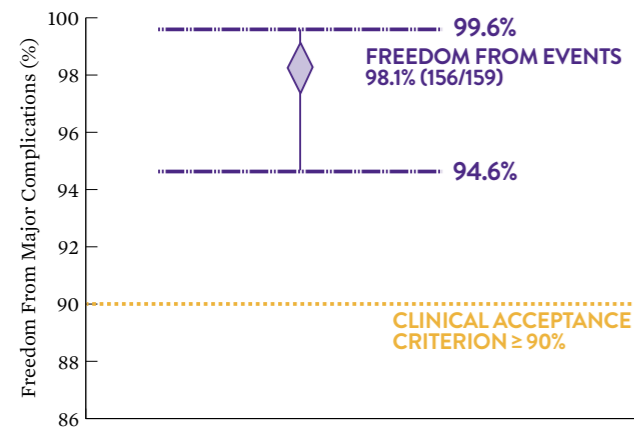

Note: Primary analysis population used and included only each subject's first occurrence of each event. Purple dotted lines represent 95% confidence intervals.

Femoral Vein Access-Related Complication Rates

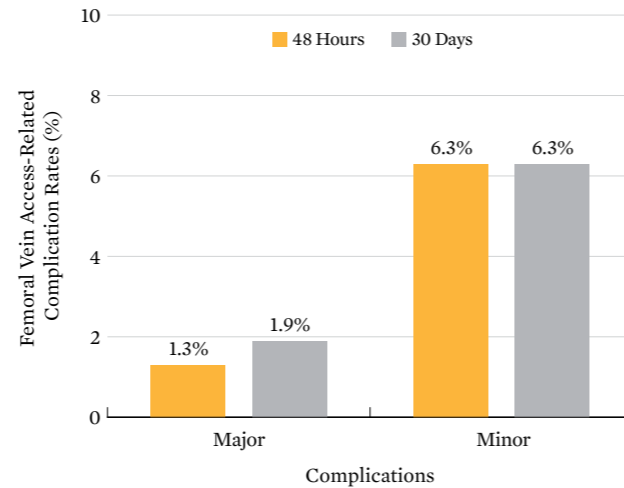

Note: Major complications were reported in 3 subjects and events included deep vein thrombosis in target limb (1), significant bleeding requiring transfusion or surgical intervention (1), hematoma not requiring transfusion or surgical intervention (1), access site re-bleeding requiring treatment or re-hospitalization (1), and pseudoaneurysm (1).

1. EVEREST II/REALISM Continued Access Registry Study.
2. The Use of the Perclose ProGlide Suture Mediated Closure (SMC) Device for Venous Access-Site Closure up to 24F Sheaths. Kar, Saibal; Hermiller, James; et al. CRT 2018.
3. Perclose ProGlide, MC, and secondary VCD (used in 1 subject [0.6%]; time to achieve hemostasis unknown).
4. Defined per Valve Academic Research Consortium (VARC)* definition as femoral vein access-related complication leading to death, life-threatening or major bleeding, requiring surgical interventions, hospitalizations, visceral ischemia or neurological impairment. The denominator excludes subjects who withdrew or lost to follow-up before the 30-day visit window (27 days post-procedure) without any femoral vein access-related complications.

*Kappetein et al. *J Am Coll Cardiol.* 2012;60(15):1438-1454.

CAUTION: These products are intended for use by or under the direction of a physician. Prior to use, reference the Instructions for Use, inside the product carton (when available) or at eifu.abbottvascular.com or at medical.abbott/manuals for more detailed information on Indications, Contraindications, Warnings, Precautions and Adverse Events. Information contained herein for **DISTRIBUTION outside of the U.S. ONLY**. Check the regulatory status of the device in areas where CE marking is not the regulation in force.

Illustrations are artist's representations only and should not be considered as engineering drawings or photographs. Photos on file at Abbott.

Abbott International BVBA

Park Lane, Culliganlaan 2B, 1831 Diegem, Belgium, Tel: 32.2.714.14.11

Perclose ProGlide and MitraClip are trademarks of the Abbott Group of Companies.

www.Vascular.Abbott

©2018 Abbott. All rights reserved. AP2946899-OUS Rev. A



DON'T JUST CLOSE.
REPAIR.

Perclose ProGlide Suture-Mediated Closure System

PROSPECTIVE
ANALYSIS
FOR LARGE-BORE
VENOUS ACCESS



TO LEARN MORE, VISIT
WWW.ABBOTTVESSELCLCLOSURE.COM/INTL

Information contained herein for **DISTRIBUTION outside of the U.S. ONLY**.
Check the regulatory status of the device in areas where CE marking is not the regulation in force.



PERCLOSE PROGLIDE COHORT IN THE REALISM¹ CLINICAL TRIAL²

A prospective analysis was performed to evaluate the safety and effectiveness of Perclose ProGlide (Perclose) in closing large-sized venous access sites through a retrospective data collection. The prospective analysis included subjects in whom Perclose was used as the primary method for large-bore venous access site closure during the MitraClip index procedure with a 24F vascular sheath.

MATERIALS AND METHODS

- 5 sites with high VCD usage and using Perclose for venous closure were included
- Primary analysis cohort (Perclose group); subjects who received at least one Perclose as the intended femoral vein access site closure device

PRIMARY ANALYSIS POPULATION


- Predefined subgroup analyses included:
 - + Perclose Alone (without any adjunctive methods other than brief [≤ 10 minutes] manual compression vs. Perclose plus secondary vessel closure method)
 - + One Perclose vs. Two Percloses

PRIMARY ENDPOINT

- Rate of freedom from major femoral vein access site-related complication at 30 days post-procedure, compared to pre-defined clinical acceptance criteria ($\geq 90\%$)

Note: Data collected at baseline, during or immediately post-procedure, at hospital discharge and at 30 days post-procedure

SITE SELECTION FLOW FOR STUDY

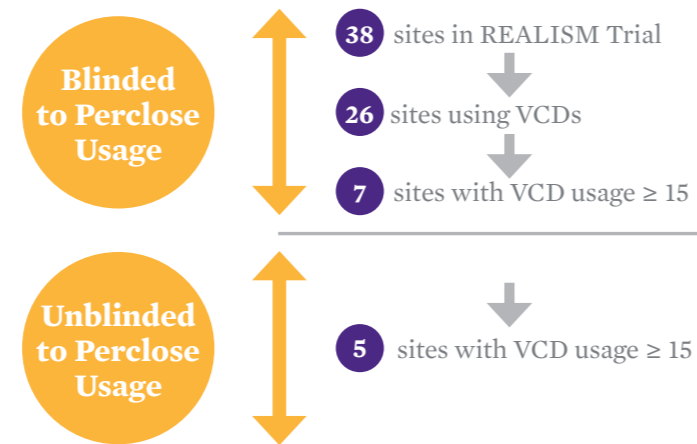


Of the 7 sites with high VCD usage, 1 did not use Perclose and 1 used Perclose for arterial access only.

BASELINE CHARACTERISTICS % (n/N)	PERCLOSE (N=159)
Age at procedure, years (range)	76 (28/98)
Male	52.8% (84/159)
Congestive heart failure	89.2% (141/158)
Hypertension	84.8% (143/158)
Atrial fibrillation	64.7% (99/153)
Coronary artery disease	67.7% (107/158)
Diabetes	26.4% (42/159)
Moderate to severe renal disease	24.5% (39/159)
Chronic pulmonary disease	23.3% (37/159)
NYHA Functional Classification III	59.7% (95/159)
IV	24.5% (39/159)
NYHA = New York Heart Association	

RESULTS

KEY STUDY FINDINGS²

- Major complication was low at 1.9%
- Freedom from major femoral vein access site-related complications was 98.1% at 30 days
- Perclose is safe and effective in the closure of venous access site with up to 24F sheath

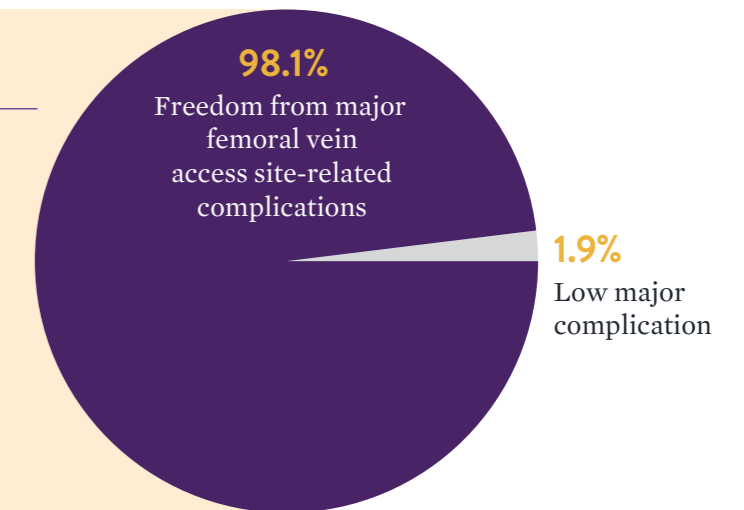


Low major complications at 30 days

SUBJECT RESULTS


- Patients treated with two Percloses 90.6% [144/159] vs. one Perclose 9.4% [15/159]
- Majority of subjects 69.2% [110/159] were treated with Perclose alone
- “Perclose Alone” group mean time to achieve hemostasis was 5.15 ± 6.05 minutes

Achievement of Hemostasis With Perclose Alone vs. Perclose Plus Adjunctive Methods

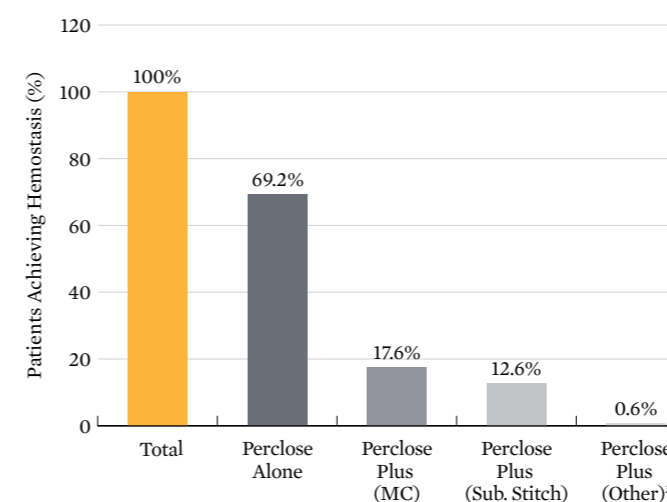

MC = manual compression, Sub. = subcutaneous.

Mean Time to Achieve Hemostasis

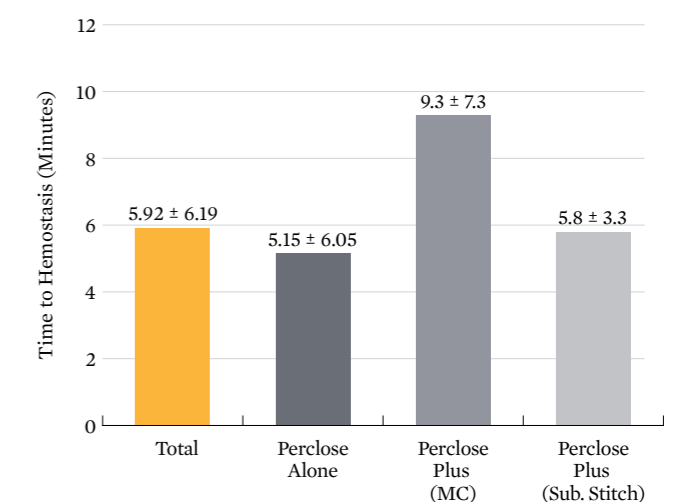

Data shows mean ± standard deviation.